

SELF - ADHESIVE SILICONE DRESSINGS & SILICONE GEL



DISPODERM® DRESSING SELF-ADHESIVE SILICONE DRESSINGS

Dispoderm® dressing is a self-adhesive silicone dressing made from medical-grade silicone and a flexible polyurethane film. These components have been selected because of their biocompatibility. Dispoderm® dressing is so flexible that the silicone dressing is easy to use on all parts of the body, including under clothing

Silicone dressings have a positive effect on:

- The appearance (height) of old and new scars
- The treatment of hypertrophic and keloid scars
- The prevention of hypertrophic and keloid scars
- The treatment of burns and donor sites

Each Dispoderm® dressing pack contains 10 silicone dressings and is sufficient for 4 to almost 8 weeks of therapy. Each silicone dressing should be used for no more than 5 to 7 days at the most.

Packaging: Box Of 10 Individually packaged

SIZE (CM)	REF
5X7	IN57
10X20	IN1020
5X30	IN530

DISPODERM® DRESSING GEL

Dispoderm Gel is a quick drying, transparent silicone gel that helps improve the appearance of scars

Total Weight: 10 gr
REF :TGS15



Contra indications of Dispoderm® dressing

Dispoderm® dressing cannot be used on:

- Open wounds
- Macerated skin

You should gradually build up the use of Dispoderm® dressing during the first week of use, until you are wearing the dressing 24 hours a day if necessary. This approach helps your skin get used to the effect of a silicone dressing.

The results of the therapy depend on how long each day you can wear Dispoderm® dressing. The longer you wear it per day, the better the results will be.

